



Baseline Set Up Instructions

You must use a standard external mouse. You may not use a finger mouse pad (ie: laptop mouse), a Track Mouse, or anything other than a standard mouse.

▶ **Minimum computer requirements:**

1. Make sure you are using Internet Explorer 6.0 and above, or Firefox 1.5 or above, and Safari for the MAC running OSX 10.2 and above.
2. You must have Macromedia FLASH PLAYER 8.0 or newer installed. You can download FLASH PLAYER at www.adobe.com
3. If you have a pop up blocker installed you must turn it off for the duration of the test.
4. Close all other programs on your computer before taking the test.
5. You need a broadband internet connection

To Take the Test

- ▶ Go to: <https://www.impacttestonline.com/stateofalaska/> and select “Launch Baseline Test”
- ▶ Enter Access Key: QZXUZZW7JM
- ▶ Click on “Launch Baseline Test.”
- ▶ The baseline test will pop-up in a new window. Click on pop-up window. If it doesn’t look like it popped up, check minimized items on the bottom of your screen.

The Test is now ready to take and set up is complete.

If you have any problems, please call Alaska Neuro Associates at 907-277-0100